

GlassOuse

Assistive Device

www.glassouse.com


Manufacturer: EnnoPro Group Limited
Address: 7th Floor, Huarong Building,
Qiaolian East, Bulong Road,
Longgang District, 518129, Shenzhen,
Guangdong, China
Phone: 0086-755-89205835(PBX)
Mail: support@glassouse.com
Web: www.glassouse.com

GW01 G-FRAMEWEAR USER MANUAL

GlassOuse G-FRAMEWEAR

G-FRAMEWEAR GW01

| | |
|--------------|---|
| Dimensions | 140mm*11mm(sunstretched state) 156mm*11mm(stretched state) |
| Weight | 21g |
| Cable Length | ABS+PC |

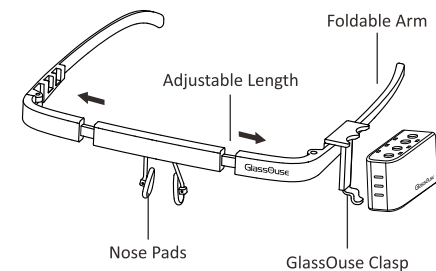

GlassOuse
Assistive Device

01

GlassOuse G-FRAMEWEAR

1. Mount the GlassOuse clasp to the G-Framewear either on the right side or left side as per your convenience.
2. Slide GlassOuse into the clasp correctly until it's firm and steady. You should hear a 'click' sound after a successful attachment.
3. Wear G-Framewear over your forehead. You can wear them over your regular glasses too.
4. Adjusting frontways length: If it's too tight or uncomfortable, hold both the handle of the frame from the front side and pull them out equally to adjust their length and vice versa.
5. To remove GlassOuse, push the remove latch and pull GlassOuse out.

GlassOuse
Assistive Device

02