

Manufacturer: EnnoPro Group Limited
Address: 7th Floor, Huarong Building,
Qiaolian East, Bulong Road,
Longgang District, 518129, Shenzhen,
Guangdong, China
Phone: 0086-755-89205835(PBX)
Mail: support@glassouse.com
Web: www.glassouse.com

GlassOuse


Assistive Device

www.glassouse.com

GW02 G-HEADBAND
USER MANUAL

GlassOuse G-HEADBAND

G-HEADBAND GW02

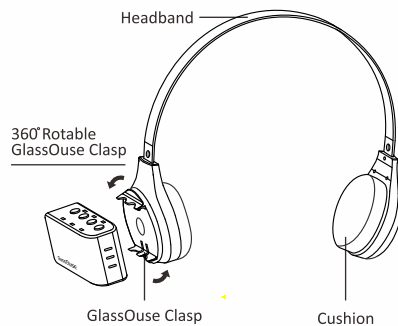

GlassOuse
Assistive Device

01

GlassOuse G-HEADBAND

Dimensions	135mm*135mm(sunstretched state) 135mm*167mm(stretched state)
Weight	66g
Cable Length	Nylon+ABS

1. Mount the GlassOuse to the G-Headband by sliding it into the clasp until they are firm and steady.
2. Wear it over your head, above your ears. The headband can be adjusted according to your size or comfort.
3. The clasp on G-headband is rotatable so that you can choose to wear GlassOuse either on the right side or left.
4. To remove GlassOuse, push the remove latch on the clasp and pull GlassOuse out.

CAUTIONS

Do not exert too much force while attaching or removing GlassOuse.

GlassOuse
Assistive Device

02