


ACCENT: 1400-30 - Changing the Battery

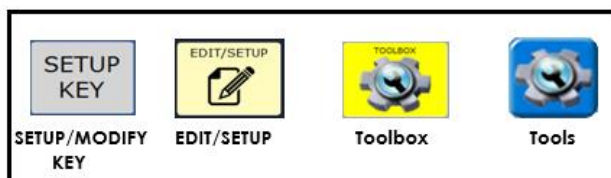

 Click [HERE](#) to download NuVoice PASS

Technical Support:

01733 370 470 (Option 2)
Email: info@liberator.co.uk

Regional AAC Consultants:


If you wish to speak to your Regional AAC Consultant, please call 01733 370 470 (Option 3)

The batteries in the new Accent 1400-30 device are now "field replaceable", so they can be changed by the user at home.

Removing the Device Frame

In order to change the battery, we first need to remove the frame and Keyguard/TouchGuide (if fitted) from the device.

1. If you are using a Keyguard or TouchGuide, remove it from the frame.
2. Using a Phillips screwdriver, remove the screws six screws that secure the frame to the device. **Important!** Set the screws aside. You will need them when you reattach the frame.

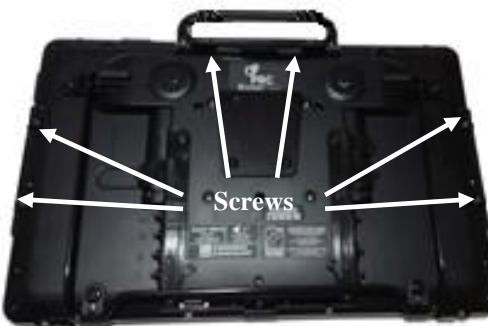

3. Pull the top of the frame loose.
4. Grasp one side of the frame and pull it loose from the device.
Hint: You may want to use a flat-head screwdriver or similar tool to gently lift the side & bottom tabs on the back of the device.
5. Pull the bottom tabs loose.
6. Grasp the other side of the frame and pull it loose from the device.

Removing the Old Batteries

1. Place the device face-down on a flat surface.



2. Remove the seven screws that hold the battery cover in place and remove the cover. **Important!** Set the screws aside. You will need them when you reattach the cover.
3. Grasp the two plastic cable connectors on either side of the opening and pull the cables out of their sockets. **Do not** pull on the wires.

Pull the connectors out of the socket



4. Using the lugs to guide you, pull the tray towards you to slide the batteries out of the device. There is a set of batteries either side. **Please contact Liberator about safe disposal of your old battery!**



Inserting the New Batteries

1. The new batteries are supplied affixed onto the trays with the cable clearly visible at the top. Slide the trays into the device.



2. Position the cable connector so that the red wires are to your right. Insert the connectors in the socket. **Important!** The battery must be positioned this way. Do not try to force it to fit differently.



3. Fit the cover back over the batteries and secure the cover with the seven screws that you previously removed.



Reattaching the Device Frame

1. Fit the top of the frame into the guide holes in the handle.
2. Pull the sides into place. **Note:** you may need to pull the bottom tabs into place as you pull on the side.

3. If your frame was originally secured with screws, reinsert the screws and tighten them.
4. Replace the Keyguard or Touchguide if you had one fitted.

PLEASE NOTE: The batteries are supplied partially charged to allow you to get your device back up and running if needed, however it is recommended that you charge the batteries fully before using your device.