


ACCENT: Connecting to an External Computer

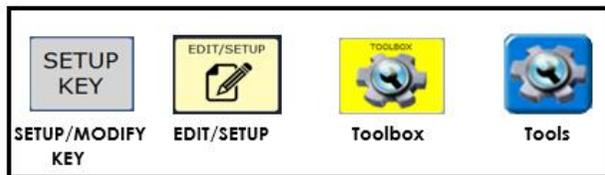

Click [HERE](#) to download NuVoice PASS


Technical Support:

01733 370 470 (Option 2)
Email: info@liberator.co.uk

Regional AAC Consultants:

If you wish to speak to your Regional AAC Consultant, please call 01733 370 470 (Option 3)

The Accent 1400/1000/800 may all be connected to an External Computer via **PRC's Bluetooth Adapter** or via a **mini USB cable**.

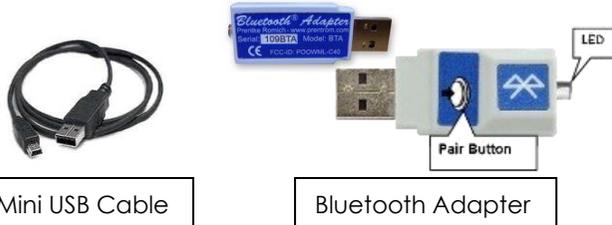

CONNECTING TO A PC VIA THE USB CABLE

Use the **USB** cable that came with your device to connect to an **IBM** OR **Macintosh** computer.

1. Turn **ON** your **computer** and **Accent** device.
2. Connect the mini-end of the **USB** cable to the mini **USB** port situated on the right side of the device.
3. Connect the other end to the **USB** port on your computer.
4. Go to **Toolbox** and select **Computer Settings**.
5. Set Output Status to **ON**.
6. Set Output Destination to **EXTERNAL**.
7. Set Connection Type to **USB**.
8. Set Output Mode to **IMMEDIATE**.
9. Leave Character Pacing **OFF**.
10. Set the Keyboard Language to **UK**.
11. Set Computer to either **IBM** or **MAC**. (Depending on what you are using)
12. Select **OK** to exit the Menu.
13. You may now open a Document within your **PC/MAC** and start using your overlay to type. You should be able to see the information appear on your computer screen.

To check at any time whether **Output** is **ON** or **OFF**, look at the status **Display Area** on your device.




CONNECTING TO A PC VIA THE BLUETOOTH ADAPTER

The **Bluetooth** adaptor has a wide range. Your communication aid and the adaptor do not have to "see" each other like infrared devices do. You can be across the room from the computer in any direction and the connection will still be made.

Important: Before you plug the adapter into your computer; Notice that the wireless adapter has a **PRC** Label on one side and a label with the **Icon** of a small switch on the other. The switch label covers a small button which is called the "**Pair Button**". Remember you will have to press this during the set-up process, so depending on where the USB ports are, you will have to access the pair button on the adapter.

1. Insert your **Bluetooth Adapter** into the **USB** port on the PC. The **LED** on the adapter may flash for a moment when you plug it in.
2. Select the **Tools Key > Toolbox > Computer Settings**.

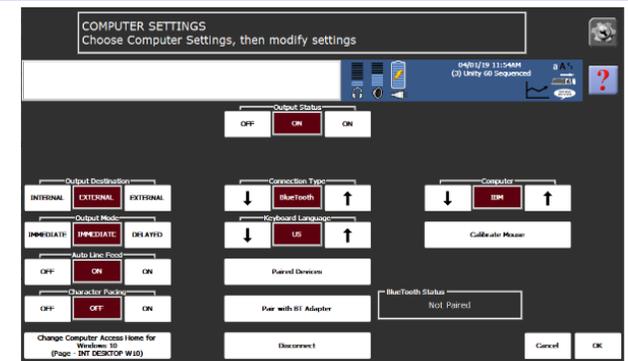

3. Set Output Status to **ON**.
4. Set Output Destination to **EXTERNAL**.
5. Set Connection Type to **Bluetooth**.
6. Set Output Mode to **IMMEDIATE**.
7. Leave Character Pacing **OFF**.
8. Set the Keyboard Language to **UK**.
9. Set Computer to either **IBM** or **MAC**. (Depending on what you are using).
10. Select the **Pair with BT Adapter** button.
11. A yellow box will appear on the screen advising to, "**Press the pair button on the adapter**". Press the **circular switch icon** on the adapter.
12. The **LED** on the adapter will flash blue. When the light changes to a solid blue, this indicates the adapter has paired with the device. **NOTE:** It can take up to 20-30 seconds to pair.
13. Once paired, the Bluetooth Status box (in the right hand corner of the Menu) will state, "**Paired, Connected**". (If the device displays "**Paired but not Connected**" go to Paired Devices, select the BT adapter and press connect). The output indicator box will be blue.
14. Select **OK** to exit the Menu.
15. Select **Go To Home**.
16. On your **PC/MAC**, open a word document or an application you can type into.
17. Now start typing using your overlay. You should be able to see the information appear on your computer screen.

****Important Note:** Once your adapter has been paired with your computer, it will always pair even if you remove it from your PC & reconnect it at a later time. The **LED** should turn blue and it should be ready to go so long as you have your settings set correctly in **Computer Settings**.