


ACCENT: Using Vocabulary Builder (LAMP WFL)



Vocabulary Builder (VB) is designed to teach new vocabulary within LAMP Words for Life UK. The 'masking' of vocabulary allows you to teach a small amount of vocabulary on an Accent device while keeping learned motor plans constant.

Vocabulary Builder (Active Side When Spelling)

When using the spell method, or when adding additional words to an existing set, you may notice the tan background changes sides. Be sure the list you are searching always has the **TAN** background. Touch the side to make it active. You would typically have the right-side **TAN** as this is the list which contains words which are masked.

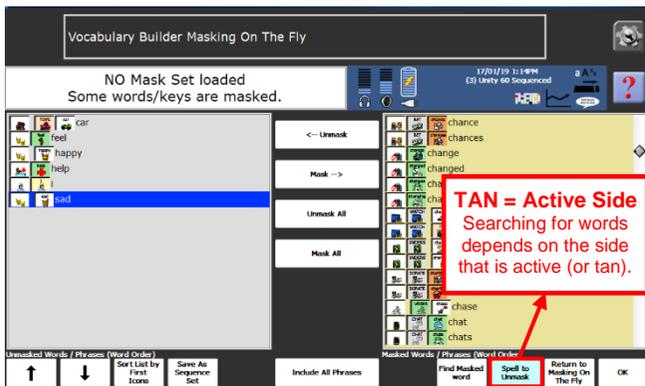

How to turn Vocabulary Builder ON/OFF

When Vocabulary Builder is ON, this icon  appears in the status display area (near the battery indicator).

1. Select **TOOLS**.
2. Select **VOCABULARY BUILDER ON/OFF**.
**Turn VB off to bring back all vocabulary quickly.



Using Vocabulary Builder to Create a Set of Visible Words

This allows you to choose the words to display and hide the remaining words.

1. Insert the USB to unlock the device if locked.
2. Select **TOOLS > VOCABULARY BUILDER MENU**.
NOTE: If you are already using Vocabulary Builder and want to unmask more words, go to step 4. If this is the **FIRST TIME** that Vocabulary Builder is being used on this device, then press **MASK ALL** in the centre column to move all vocabulary from left to right.
3. Make sure the right-hand side of the screen is tan coloured by tapping it (see picture below left)
4. Select **SPELL TO UNMASK**.
5. Type word(s) to unmask, using commas to separate words and/or phrases e.g.
go,more,stop,play.
6. Select **OK** several times to complete.

NOTE: All the words you have chosen will now appear in the left grey column. If some of the word(s) are missing this is likely because the word(s) are not stored in the vocabulary.

How to Save a Sequence Set

If you want to keep a selection of word(s) you have built (known as a Sequence Set), you need to **SAVE** them.

1. Select **TOOLS > VOCABULARY BUILDER MENU**.
2. Select **SAVE AS SEQUENCE SET**.
3. Type the name for the Sequence Set.

4. Select **OK** twice to exit.

NOTE: You may save as many sequence sets as you wish. This is particularly useful if you are using the Accent device with multiple students, or if you want to retrieve a set of words quickly.

How to Load a Sequence Sets

After creating a Sequence Set, it can be retrieved from the Vocabulary Builder Menu.

1. Select **TOOLS > VOCABULARY BUILDER MENU**.
2. Select **ADVANCED OPTIONS**.
3. Select **USE SEQUENCE SET**.
4. Select **LOAD SEQUENCE SET**.
5. Choose the required set. Select **OK** twice.

Adding a NEW word

1. Go to the page where you wish to add your word.
2. Select the **TOOLS > EDIT/SETUP KEY**.
3. Select the button where you want to add the new word.
4. Select **CHANGE PICTURE > SPELL ICON TO FIND**.
5. Type the name of the picture you want. Select **OK**.
6. Choose a picture.
If you want the spoken message to be different:
 - a. Select **ADD MESSAGE OR INSERT TOOL**.
 - b. Select **CLEAR**.
 - c. Type the new message – remember a space at the end!
 - d. Select **OK**.
7. Once finished, select **OK** to complete.

ACCENT: Creating Smart Charts Using the NuVoice Emulation Software (PASS)

A “**Smart Chart**” is based on a set of words created using the **Vocabulary Builder feature** in the **Accent** devices. **Vocabulary Builder** is a therapeutic tool that allows you to easily teach new vocabulary within **LAMP Words for Life**. Using the **Emulation Software** on your computer, available from the **Liberator Support** website, you can create support materials to accompany **Vocabulary Builder** and assist with teaching vocabulary in the device.



Click [HERE](#) to download NuVoice PASS

Technical Support:

01733 370 470 (Option 2)

Email: info@liberator.co.uk

Regional AAC Consultants:

If you wish to speak to your Regional AAC Consultant, please call 01733 370 470 (Option 3)

To create a smart chart you must use **Vocabulary Builder** to create the set of words you wish to teach. See page 1 of this sheet.

Before you can create your smart chart you must **SAVE** your vocabulary on the Accent device and **LOAD** it into the NuVoice emulation software. To do this:

SAVING VOCABULARIES

On the Accent device, save the vocabulary that you want to add to the NuVoice emulation software (PASS) on your PC.

1. Insert a USB stick into the Accent device.
2. Select **TOOLS > TOOLBOX > IMPORT/SAVE...**
3. Select **SAVE A VOCABULARY ***.
4. Select your vocabulary, indicated by (current area).
5. Select the USB stick as the destination, then press **OK**.
6. Type the name of the vocabulary, e.g. Client nameDDMMYY.

7. Press **OK** and wait for save to complete, once complete select **OK** and remove the USB stick.

* **NOTE:** Selecting “Save A Vocabulary” will NOT save all the settings associated with that vocabulary, such as IR, computer settings, etc. To ensure that these are saved select “Save ALL Vocabularies” instead.

IMPORTING VOCABULARIES

Import the vocabulary that you saved onto the USB stick from the Accent device into the NuVoice emulation software (PASS).

1. Ensure the NuVoice emulation software (PASS) is open on your PC and insert a USB stick into the PC.
2. In the software, select **TOOLS > TOOLBOX > IMPORT/SAVE...**
3. Select **IMPORT A VOCABULARY ***
4. Choose where you want to import the vocabulary to. Select the USB stick as the source.
5. Select the vocabulary you wish to import.
6. Once imported, press **OK** several times to exit.


* **NOTE:** If you selected “Save ALL Vocabularies” in the previous section, ensure you select “Import ALL Vocabularies” to import into the emulation software (PASS).

You should now have the personalised vocabulary added to your NuVoice emulation software (PASS).


CREATING A SMART CHART

With your personalised vocabulary imported and on screen you need to do the following:

1. Select **Vocabulary > Vocabulary Display Options > Create Smart Chart**.

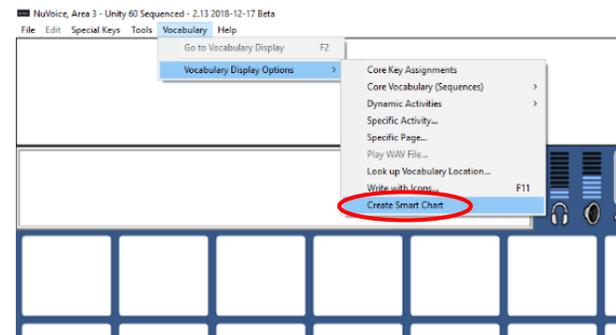

2. You will now see a screen that looks like this:

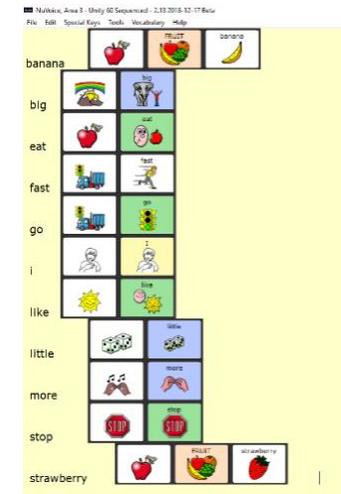

3. Select **File > Save and Open**
4. Choose the destination folder to save your Smart Chart
5. Type the name for your Smart Chart (e.g. Marks Words). This will now open in a Word Document.
6. Within the Word document, select CTRL and A to highlight ALL the words.
7. Select **Insert > Table > Insert Table**. This will now put your words into a table and align the words alphabetically.
8. Select **File > Save As** and save as you would any document. You may want to save it as a PDF file too as it is a smaller file size to email.