




NovaChat & LR7 - LAMP Words for Life (WFL) Quick Reference Guide



Important Functions

Below are the main Functions buttons on the NOVA chat devices:

Menu Button  /  /  allows you to determine which vocabulary file will be used, which augmentative options are enabled, how and when to speak messages, and how a wide variety of other options are configured.

Home Button  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  /  / /

Word Finder

1. To turn on Wordfinder for the first time select **Menu > Application Settings > Word Finder**. Select **Show Menu Item** (green tick). If Word Finder is already turned on go to step 3.
2. Press back until you return to the vocabulary.
3. Select **Menu > Word Finder**.
4. Type the word you want to find. If the word is stored in the device, the corresponding icon sequence will appear.
5. Press **Back** until you reach WFL home.

Edit/Create a Button

1. Select **Menu > Edit Mode**.
2. Select and hold the button you want to edit.
3. Select **Edit Button** if the button already has something on it. Select **Create Button** if the button is blank.
4. Enter the text you want in the Label and Message boxes respectively (tap in the appropriate box to bring up the keyboard for entering text).
5.
 - a. Select **Find** to select a pre-stored symbol. Tap in the search bar at the top, type an image name, select **Search**, and search by category.
 - b. Select **Import** or **Camera** to use a photo (see "Add a Photo to a Button" below).
6. Select the **Actions** tab across the top.
7. In WFL, if the button should speak there should already be a command for Speech Message, select **Save** to finish. If the button will link to another page, see "Link a Page to a Button" and continue from step 5.

Add a Photo to a Button

If the photo is already in the camera roll:

1. Select **Menu > Edit Mode**.
2. Select and hold the button you want to edit.
3. Select **Edit Button** if the button already has something on it. Select **Create Button** if it is blank. The Button Properties used to edit the button will be displayed.
4. Select **Import** and select the photo you want (you may need to select the picture twice).
5. Rename the photo if needed. Select **Save** twice.

If the photo is NOT in the camera roll:

1. Select **Menu > Edit Mode**.
2. Select and hold the button you want to edit.
3. Select **Edit Button** if the button already has something on it. Select **Create Button** if it is blank. The Button Properties used to edit the button will be displayed.
4. Select **Camera**. Frame the photo and then select the **Camera** icon to take the photo. Wait.
5. Select **Save** (top left) or press the **tick**.
6. Rename the photo if needed. Select **Save** twice.

Link a Page to a Button

1. Select **Menu > Edit Mode**.
2. Select and hold the button you want to edit.
3. Select **Edit Button** if the button already has something in it. Select **Create Button** if it is blank.
4. Select the **Actions** tab across the top.
5. Select **Add an Action**.

6. Select **Navigate** if you want the page to remain open after you speak a word. Or, select **Jump to Page** to close the page after you speak a word.
WFL recommends "Jump to Page".
7. To create a page, do either of the following:
 - Select **Template**, select **LAMP 84 Template**, select **OK**, and enter a new page name.
OR
 - Select **New**, enter the page name, and select **7** rows and **12** columns.
8. Select **Save**.
9. Remove the Speech Message action by pressing and holding **Speech Message** and then selecting **Remove**.
10. Under the **General** tab, add an image & label to the button.
11. Select **Save**.

Change Skin Tone in SymbolStix Vocabularies

This new feature lets you set single or randomised skin tones in individual SymbolStix libraries (available in version 2.30 & above).

1. Select **Menu > Edit Mode**.
2. Select **Menu > Vocabulary Settings > SymbolStix Settings**.
 - a. To use a single skin tone select **Skin Tone** and choose a skin colour. All symbols will have this same skin colour.
 - b. To use random skin tones enable **Use Random Skin Tones** (green tick), then select **Randomize Skin Tones**. A message will appear stating that this cannot be undone – select **Yes**. If you wish to revert back to a single skin tone deselect **Use Random Skin Tones** and follow step a above.
3. Press **Back** until you reach WFL home.

Transfer Files between the Device and Editor

NOTE: Any vocabularies saved from software version 2.34 or higher (either in Chat Editor or Chat Software) will not import into a lower software version.

1. Start Chat Editor.
2. Connect a USB cable to the device and the computer that is running the Editor. The device screen will display "Transfer Mode".
3. From the Editor, select the **Library** menu and then click **Transfer**.
4. From the Transfer screen click **Vocabs to/from Device**.

Hint: In order for the transfer process to work, the NOVA chat and the Chat Editor software MUST be running the same version software. To check the device software version choose **Menu > Help > Check for updates**. To check the Chat Editor software version select **Help > About**. You will need to perform a software update if there is a mismatch.

Copy Files from the Device to Chat Editor

1. From the Transfer screen, select the vocabulary file(s) you want to transfer from the device to computer.
2. Click the **To Editor** button.
3. When the transfer is complete, the name of the file(s) should be listed in the Chat Editor section of the Transfer screen.
4. Click **Done**.

5. When you are finished transferring files, disconnect the USB cable from the device and the computer and return to NOVA chat.

Copy Files from Chat Editor to the Device

1. From the Transfer screen, select the vocabulary file(s) you want to transfer from the Chat Editor to the NOVA chat device.
2. Click the **To Device** button.
3. When the transfer is complete, the name of the file(s) should be listed in the Vocabs on Device section of the Transfer screen.
4. Click **Done**.
5. When you are finished transferring files, disconnect the USB cable from the device and the computer and return to NOVA chat.

How to use KIOSK mode & Menu Lock

Kiosk mode will stop the user accessing the Android system. Menu Lock prevents the user from changing any settings within LAMP WFL.

Using the KIOSK mode setting:

1. Select **Menu > Application Settings > System > Kiosk Mode**.
2. If you wish to set a password check the **Require Password** box, enter a new password, then select **Save** *. (Remember your password).
3. Place a check mark in the box next to **Kiosk Mode**.
4. Press **Back** until you reach WFL home.

* If you don't want to set a password then skip step 2.

Lock/Unlock the Menu

To lock the menu:

1. Select **Menu > Application Settings**.
2. Select **Menu Lock**. Select **Menu Lock** again.
3. Enter a password. Confirm the password.
4. Select **Save**. (Remember your password).
5. Press **Back** until you reach WFL home.
6. Select **Menu > Lock**.

To unlock the menu:

1. Select **Menu**.
2. Select **Unlock**. Enter your password and select **OK**.

NOTE: Enabling Menu Lock as well as Kiosk Mode will add another level to keep the user inside Nova Chat. By applying the Menu Lock, no one will be able to get into application settings to turn off KIOSK mode.