

Transferring your vocabulary from LAMP WFL (old) to LAMP WFL (new)



LAMP WFL App (Old)

1. Select **Vocab**>**Choose New Vocab**

15:43 Tue 24 Jun 97%

Vocab		Menu									
Choose New Vocab		up	yes	good	some	no	down	out	off	bad	
me	my	wear	am	please	that	and	in	what	a	+s	there
I	we	are	is	were	was	on	to	SPELL/NUM	an	the	end
you	they	new	play	like	work	have	feel	read	more	fast	stop
it	he	want	all	come	time	do	go	get	big	color	help
she	look	slow	hear	think	right	said	live	love	follow	ride	put
CLEAR	not	talk	sit	eat	find	make	need	drink	watch	turn	sleep


2. Select **Menu>Import/Export Vocab**




3. Select **Export and Share**




4. Select your customised vocabulary from the menu

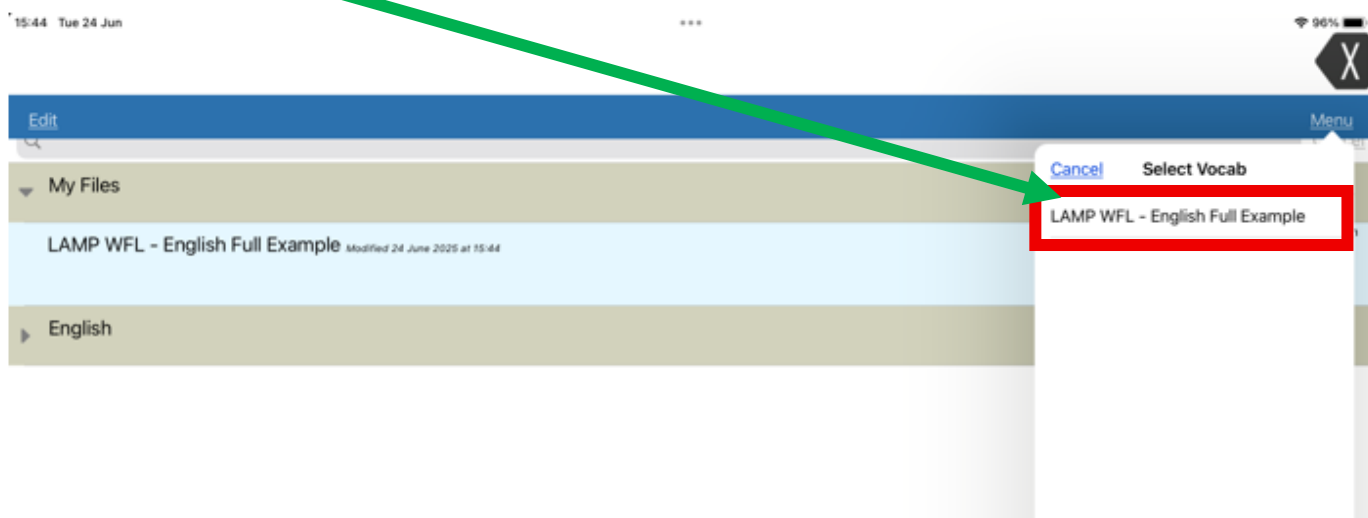


5. Select **Done**

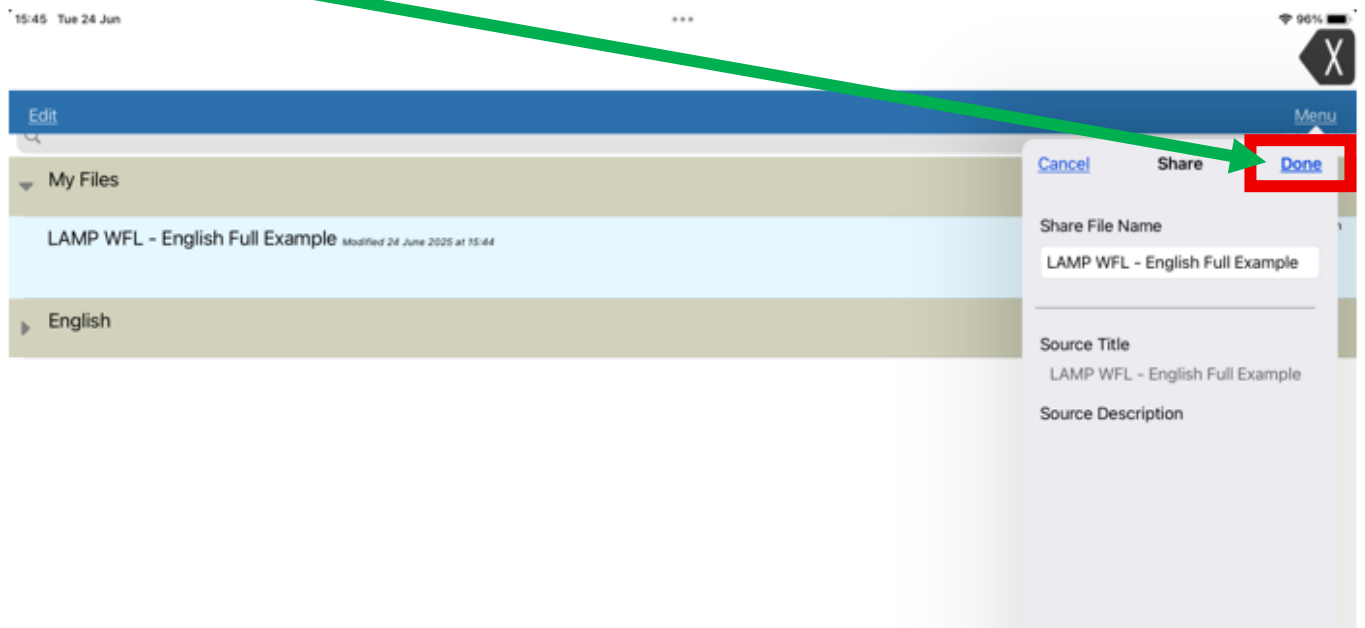


6. Select **LAMP WFL (New)** – Note: you may need to swipe from right to left to locate the app.

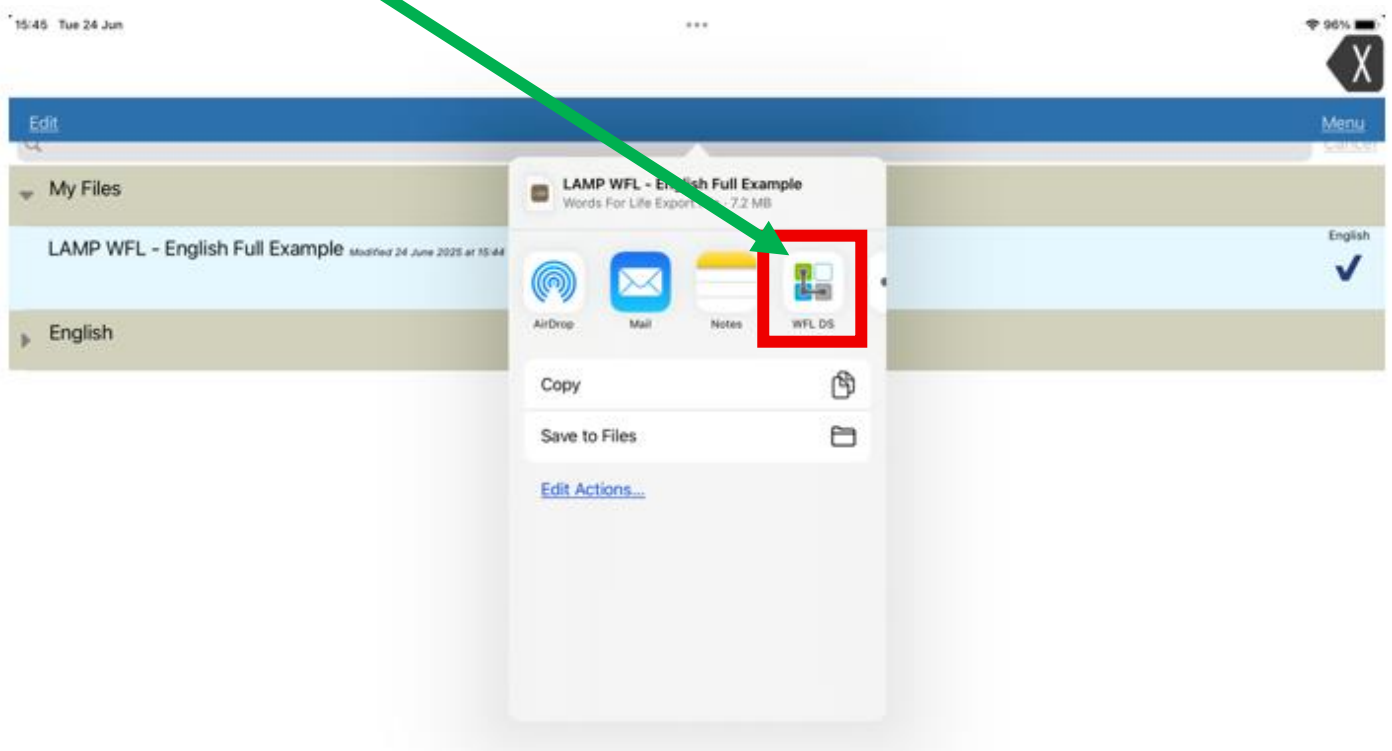

7. Select **Yes**

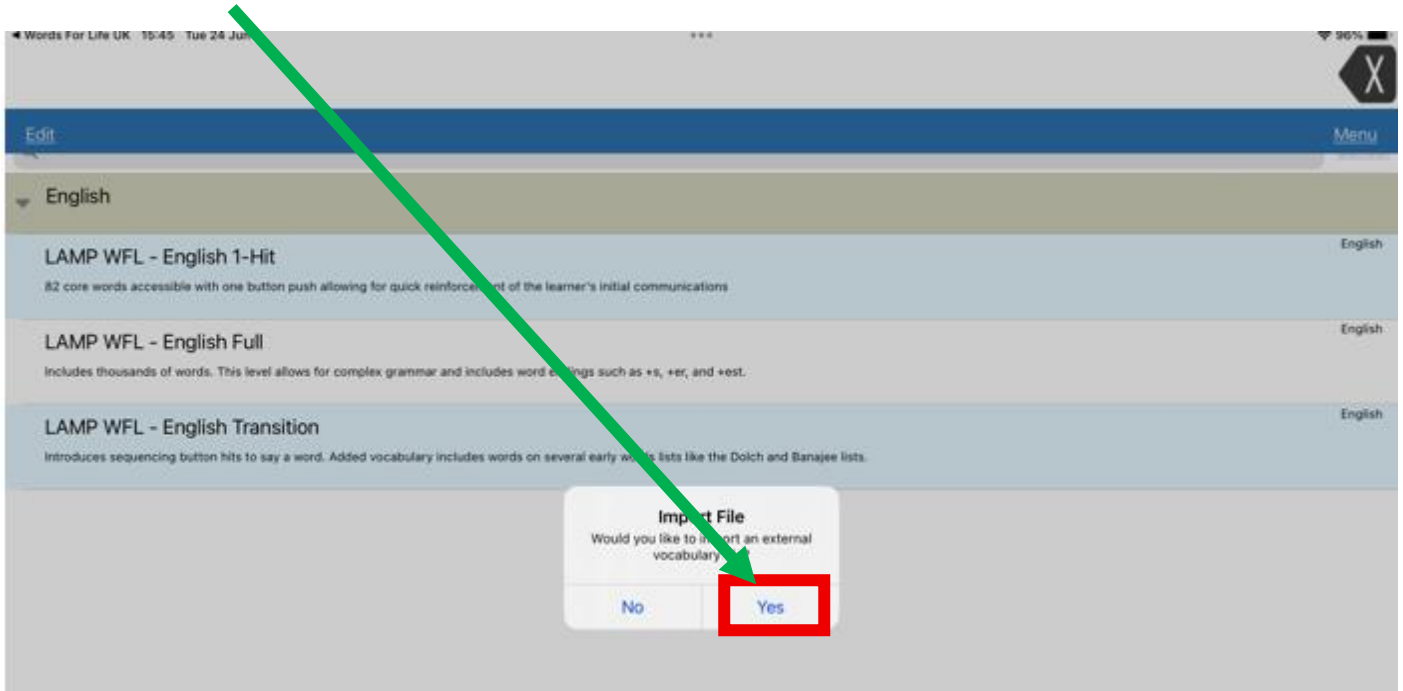

8. Select **Save**

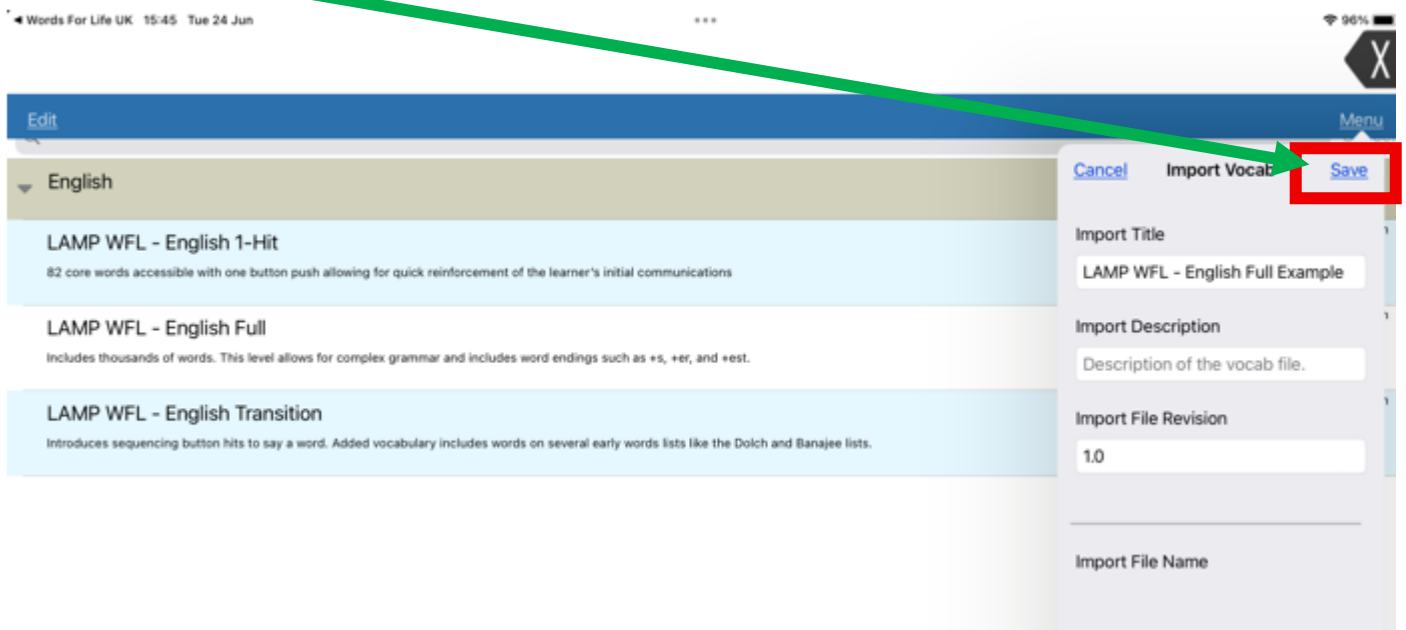

Your customised vocabulary has now been transferred to LAMP WFL (new)

