

LAMP (WFL): Transferring Vocabularies from a NovaChat/LR7 to an iPad

Click [HERE](#) to download the Chat Editor Software.

Technical Support:

01733 370 470 (Option 2)

Email: info@liberator.co.uk

Regional AAC Consultants:

If you wish to speak to your regional consultant, please call 01733 370 470 (Option 3)

Backing up to Chat Editor from a NovaChat/LR7 Device

To transfer files between the device and Chat Editor, do the following:


1. Ensure that the Chat Editor CD software is installed on the PC.
2. Connect the device to the PC with a USB cable.
3. Start the Chat Editor software.
4. In the Editor, select **Library** to open the Resource Browser.
5. From the Resource Browser, select **Transfer**, and then select **Vocabs To/From Device**.
6. From the Transfer screen, select the vocabulary file or files you want to transfer.
7. Click the **To Editor** button.
8. When the transfer is complete, the file name will be listed in the Editor files.

Transferring Vocabularies from Chat Editor to iTunes

In order to access your vocabulary on your iPad you will need to transfer the file from Chat Editor into your iTunes account:

1. Connect your iPad to your PC using the USB cable.
2. Open Chat Editor on your PC.
3. Select **Library**, then click on **Transfer**, then **Vocabs to File**.
4. Select your vocabulary then click **Transfer**.
5. Select where you want to save your file, rename it if you wish, then click **Save**.
6. Close the Transfer window & Chat Editor.
7. Open iTunes on your PC.

8. Select the small device icon

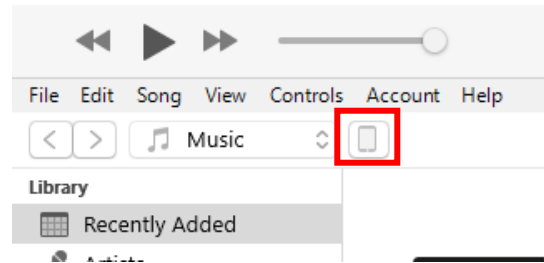

9. Select **File Sharing** on the left then choose **Lamp Words for Life** from Apps column.
10. Select **Add File**, navigate to your saved vocabulary file and click **Open**.
11. Your file is now added into iTunes
12. Click **Sync**, then **Done**.

Opening a Transferred Vocabulary

Now that you've transferred your vocabulary to your iPad you will need to import it into your Lamp WFL app on the device:

1. Select **Vocab**, then **Choose New Vocab**.
2. Select **Menu**.
3. Select **Import/Export Vocab**.
4. Select **Import from iTunes**.
5. Select your vocabulary, change the import title & description if you wish, then tap **Save**.
6. Once imported, select your vocabulary from the list to open.


Backing up a Customised Vocabulary File using iTunes

It's a good idea to keep a copy of all custom files on a computer for safekeeping.


1. Connect your device to a computer using a USB cable.
2. Open iTunes on the computer.
3. In Lamp Words for Life tap **Vocab > Choose New Vocab**.
4. Tap **Menu > Import/Export Vocab > Export to iTunes**.
5. Select the vocabulary file to export and select **Save**.
6. In iTunes on the computer, click the small device icon in the upper left portion of the window.
7. In the window that opens, click **File Sharing** on the left.
8. In the Apps column, select the **Lamp Words for Life** app.
9. From the column under **Words for Life Documents**, select the vocabulary file you are exporting.
10. Click **Save** on the bottom right. A Save window will open.
11. Navigate to the folder in which you want to save the file and click **Select Folder**.
12. Click **Done** and exit from iTunes.

NOTE: Should you wish to save a vocabulary or word list from your iTunes account to your PC this process can be followed in reverse:

1. Connect your iPad to your PC using the USB cable.
2. Open iTunes on your PC.
3. Select the small device icon

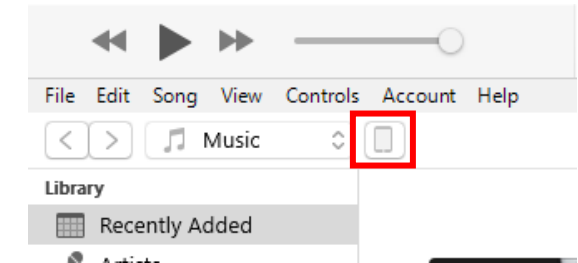

4. Select **File Sharing** on the left then choose **Lamp Words for Life** from Apps column.
5. Select the vocabulary/word list you wish to save and click **Save**, navigate to your saved vocabulary file and click **Select Folder**.

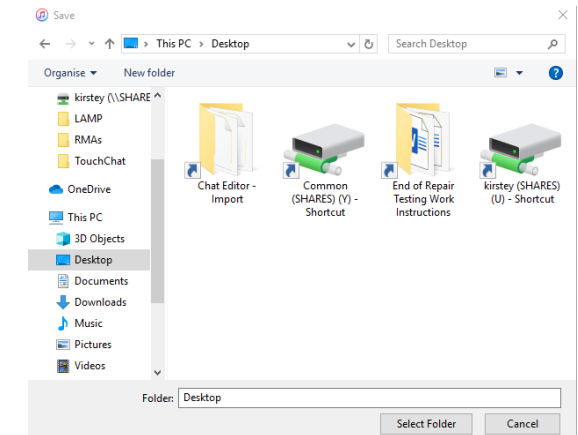

6. Your file is now saved to your PC.