



LR7-20+ RUGGED - TRANSFER VOCABULARY FILES TO AND FROM A USB STICK




Important Functions

Below are the main Functions buttons on the LR7 Rugged devices:

Menu Button  allows you to determine which vocabulary file will be used, which augmentative options are enabled, how and when to speak messages, and how a wide variety of other options are configured.

Home Button  navigates out of Chat software to the Home screen of the Android software.

Back Button  navigates to previous page or to previous Menu. Pressing and holding the Back button also reveals the Chat software menu.

 Click [HERE](#) to download the Chat Editor Software.

Technical Support:

01733 370 470 (Option 2)
Email: info@liberator.co.uk

Regional AAC Consultants:

If you wish to speak to your Regional AAC Consultant, please call 01733 370 470 (Option 3)

You may backup to and from a USB stick if your device is **running v2.15 software and above**. This is true for all LR7 RUGGED devices (20-series & up).

You will require a USB Stick and a USB C to USB OTG adapter. These are provided when you purchase the LR7-20/30.




The USB stick plugs into the OTG adapter which is then plugged into the device - see below:




Transferring Vocabulary Files

1. Plug in the USB stick into the USB C to USB OTG adapter.
2. Connect the OTG adapter into the USB C port on your device.
Note: If the device's file manager opens, use the device's Back button to close it.
3. Change the default write disk to External USB Storage (a pop-up message will prompt you to change this).
4. In Chat Software, select **Menu > Library**.
5. Select **Menu > Import/Export**.

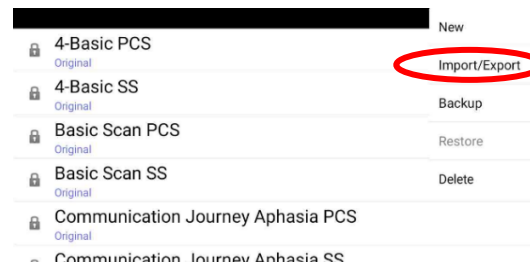


A different file management screen may open. Depending on your device, select or navigate to the USB/External drive, choose the folder you want, and choose Select or OK.

6. Select the **LOST.DIR** folder if using the Saltillo USB stick.

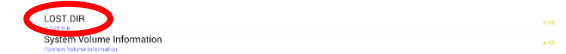


A window will list the customized files under **Device** on the left. Any files already on the USB drive are listed under **Backup Folder** on the right.

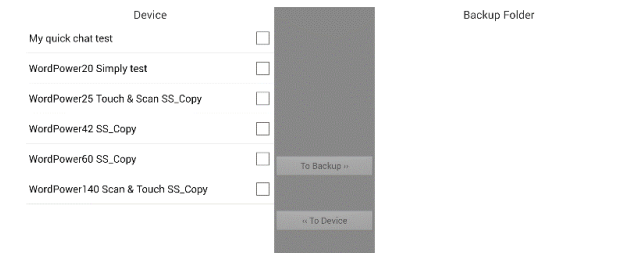


To transfer files from your device to the USB drive:

1. Select each file on the device that you want to copy to the USB drive.
2. Select **To Backup**, which is now highlighted. The files will be copied to the USB drive. When the files finish copying, they will be listed under **Backup Folder**.

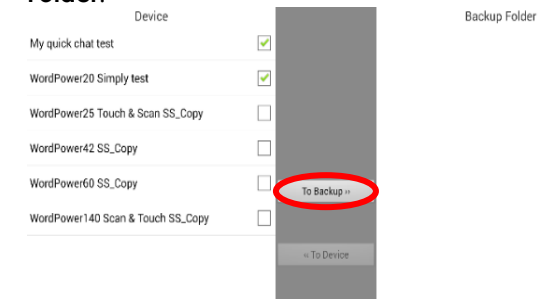

3. Your backed-up files are displayed in the **Backup Folder**.

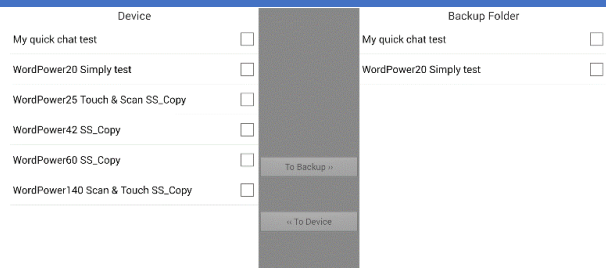


Typically, if an OTG adapter is used with YOUR own USB stick, the setup will look as shown below:



To transfer files from the USB drive to your device:

1. Select each file on the USB stick that you want to copy to the device.
2. Select **To Device**, which is now highlighted. The file(s) will be copied to the device. When the files finish copying, they will be listed under **Device**.
3. When you have finished transferring files, disconnect the USB stick from your device and store it in a safe location and/or copy the files from the USB drive to a computer for safe keeping.

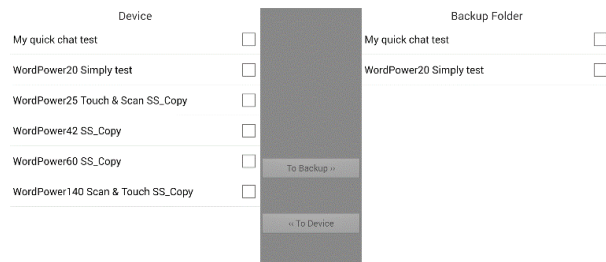

USB Transfer – Known Errors

If you are importing a file that contains an imported image, and that same imported image also exists on two files on the device, the transfer will error out.

Using Your Own USB Stick - Specifications

1. Format your stick to:
 - FAT 32
 - Use Windows Quick Format
2. Your USB stick:
 - Must confirm the shaft is long enough to get through the boot and shells
 - On the Go (OTG) cables – shaft length, 4th and 5th pins need to work for USB transfer and not just charging