



Liberator

Words to model - easyChat Core 60 WS

Word	How to say it	Word	How to say it
Again	LFL WORDS + again icon	little	DESCRIBE little icon
bad	DESCRIBE bad icon	look	ACTIONS + look icon
big	DESCRIBE big icon	me	me icon
close	ACTIONS + close icon	mine	mine icon
come	come icon	my	my icon
different	DESCRIBE different icon	off	POSITIONS off icon
do	do icon	on	on icon
don't	don't icon	play	play icon
down	POSITIONS down icon	put	ACTIONS + put icon
fast	DESCRIBE fast icon	ready	DESCRIBE ready icon
feel	feel icon	sad	DESCRIBE sad icon
get	get icon	slow	DESCRIBE slow icon
go	go icon	stop	stop icon
good	DESCRIBE good icon	take	ACTIONS + take icon
happy	DESCRIBE happy icon	that	THIS THAT THESE THOSE that icon
help	help icon	turn	turn icon
I	I icon	want	want icon
it	it icon	what	QUESTIONS what icon
like	like icon	you	you icon